FFP2

**FIRST<sup>®</sup>  
PROTEC**

**small**

YPHD : YD-006 FFP2 NR EN 149:2001+A1:2009

TALLA  
PEQUEÑA

SMALL SIZE

# FICHA TECNICA

## YPHD YD-006

**FFP2 NR**  
Media Máscara Filtrante

**CE | 1463**

UNE EN 149:2001+A1:2009

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**ENG.- SEE INFORMATION SUPPLIED BY THE MANUFACTURER.**  
Method of use. Step 1. Open the mask, with the nose clip up, and pull one ear strap with each hand. Step 2. Hold the mask on the face and hold the mask against your chin. Step 3. Fit the two ear loops on your ears separately. Step 4. Place the fingers above the top of the mask and use your thumb to press and move the nose clip to the side of the nose and lip. Do not press on the strap or bridge.  
INSTRUCTIONS. Please check whether the package is in good condition. If damaged, do not use it. Check the information on packaging and make sure the product within the validity date. Product do not reuse. Each mask should not be used for more than 12 hours. Working time should not exceed 12 hours in a row. Do not use the mask during a shift. Do not reuse the mask. Please refer to the Safety Data Sheet. 2. Before placing it on the ears, secure the nose clip and secure the mask. This product is designed to protect against both solid and liquid aerosols from the ambient atmosphere. It has passed EN149:2001+A1:2009 in accordance with the CEI of Regulation (EU) 2016/425.  
Modification Scope. Suitable for daily protection during high seasons such as flu, haze and pollen allergy.  
Production Date. Printed on the package. Production Batch No. Printed on the package. End of Shelf Life: 2 years.  
Manufacturer / Fabricante en PRC: Guangdong Yida Medical Producting Co., Ltd.  
Room 502, Building No.1, Lane 1, Xian Road Hengfeng, Dongguan City, Guangdong Province.  
P.O. China MODEL: YD-006 (FPM)

[www.firstprotec.com](http://www.firstprotec.com)  
Imported / Importada por Diemark Products S.L., Val Edison 19 C,  
Sant Joan de Vilatorrada 15105, La Coruña Spain.  
info@firstprotec.com

Read the instruction manual  
carefully before use  
LEAER  
MADE IN CHINA

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**ENG - SEE INFORMATION SUPPLIED BY THE MANUFACTURER.**  
Method of Use: Step 1, Open the mask, keep the nose clip up, and pull one ear strap with each hand. Step 2, Hold the mask on the face and hold the mask against your chin. Step 3, Put the two ear belts on your ears separately. Step 4, Place the fingers above the top of the built-in metal bar, press and move the finger tips to the side of the nose until the clip is pressed into the shape of a bridge.  
**INDICATIONS:** Please check whether the package is in good condition. If damaged, do not use it. Check the information on packaging and make sure the product within the validity date. Product do not reusable. Each mask should not be used for more than 1 shift. Replacing mask between 12 hours is recommended. Wash your hands carefully after handling used mask. Condition of Storage and Transportation: Well-ventilated and Non-corrosive Storage. Protection from moisture and sunlight during transportation. Storage Temp: -10°C +40°C. Storage Humidity: <80% RH.  
**Description of Product Functions and Compliance:** This product is designed to protect against both solid and liquid aerosols from the ambient atmosphere, it has passed EN149:2001+A1:2009 to show conformity to the EHSR of Regulation (EU) 2016/425  
Application Scope: Suitable for daily protection during high seasons such as flu, haze and pollen allergy.

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Read this instruction manual carefully before use  
**MADE IN CHINA**

Fecha de producción/ Lote / Final de vida útil: 2 años.  
Produktion/Date/Batch No.: End of Shelf Life: 2 years.  
Data de fabrica / Nº de lote / Utilizar num prazo máximo de: 2 anos.  
Produktionsdatum / Chargennummer /Haltbarkeitsende: 2 Jahre.  
2021010101 / 2021.01.01 / 2022.12.31

<p><b>FFP2 NR</b></p> <p>Filtering half mask Media Máscara Filtrante NR</p> <p><b>FIRST<sup>®</sup> PROTEC</b></p> <p><b>small</b></p> <p>YPHD <b>YD-006</b></p> <p><b>2000 PCS</b> FIRSTPROTEC.COM <b>CE 1463</b></p> 	 <p>Description: FFP2 Filtering half mask Notified Body: CE 1463 Standard: EN149:2001+A1:2009 MODEL NO.: YD-006 QTY: 2000 Pcs G.W.: 11.5kgs N.W.: 7.6kgs SIZE: 57x52x34.5cm</p> 	<p><b>FFP2 NR</b></p> <p>Filtering half mask Media Máscara Filtrante NR</p>  <p>Manufacturer / Fabricado en PRC por: Guangdong YIDao Medical Technology Co., Ltd. Room 302, Building 2, No. 1, Lane 1, Xiju Road Hengli, Dongguan City, Guangdong Province, PR China   MODEL YD-006   YPHD</p> <p><b>FIRSTPROTEC.COM</b> Imported / Importado por Dismark Products SL Via Edison 19 C., Santiago de Compostela 15705, la Coruña Spain. info@firstprotec.com</p> <p><b>MADE IN PRC</b></p>	<p><b>FFP2 NR</b></p> <p>Filtering half mask Media Máscara Filtrante NR</p> <p><b>FIRST<sup>®</sup> PROTEC</b></p> <p><b>small</b></p> <p>YPHD <b>YD-006</b></p> <p><b>2000 PCS</b> FIRSTPROTEC.COM <b>CE 1463</b></p> 

**Fi** Album  
YPHD YD-006

**FIRST<sup>®</sup>**  
**PROTEC**