10 PACK

**FIRST[®]
PROTEC
FFP2**

FICHA TECNICA
ecomatters JY-MF-B1

FFP2 NR
Media Máscara Filtrante

CE | 0598

UNE EN 49:2001+A1:2009

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EAN 13

EcoMatters JY-MF-B1
CE 0698 FFP2 NR
EN 149:2001+A1:2009

Please refer to the instruction before use.

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USE THE INSTRUCTIONS SUPPLIED BY THE MANUFACTURER.
 This product is designed to be used in ambient temperature conditions between 5 °C and 50 °C, and relative humidity between 40% and 80%. It is not recommended to use this product in conditions of temperature above 50 °C or below 5 °C, or in conditions of humidity above 80% or below 40%.

READ THE INSTRUCTIONS SUPPLIED BY THE MANUFACTURER.
 This product is designed to be used in ambient temperature conditions between 5 °C and 50 °C, and relative humidity between 40% and 80%. It is not recommended to use this product in conditions of temperature above 50 °C or below 5 °C, or in conditions of humidity above 80% or below 40%.

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ENG - SEE INFORMATION SUPPLIED BY THE MANUFACTURER.

Method of Use: Step 1. Open the mask, keep the nose clip up, and pull one ear strap with each hand. Step 2. Hold the mask on the face and hold the mask against your chin. Step 3. Put the two ear belts on your ears separately. Step 4. Place the fingers above the top of the built-in metal bar, press and move the finger tips to the side of the nose until the clip is pressed into the shape of bridge.

INDICATIONS: Please check whether the package is in good condition. If damaged, do not use it. Check the information on packaging and make sure the product within the validity date Product do not reusable. Each mask should not be used for more than 1 shift. Replacing mask between 12 hours is recommended. Wash your hands carefully after handling used mask. Condition of Storage and Transportation: Well-ventilated and Non-corrosive Storage. Protection from moisture and sunlight during transportation. Storage Temp: 5°C - 50°C. Storage Humidity: ≤80% RH.

Description of Product Functions and Compliance: This product is designed to protect against both solid and liquid aerosols from the ambient atmosphere, it has passed EN149:2001+A1:2009 to show conformity to the EHSR of Regulation (EU) 2016/425

Application Scope: Suitable for daily protection during high seasons such as flu, haze and pollen allergy.

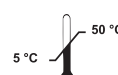
Product Structure: (5 layers) 1st ply: Polypropylene spunbond non-woven fabric, 2nd ply : Filter cotton, 3rd&4th ply : Meltblown non-woven fabric,5th ply : Polypropylene spunbond non-woven fabric.

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